

Bonopty[®]

BONE BIOPSY SYSTEM 14 GAUGE

Optimize tactility and efficiency during your procedures with Bonopty[®] products



Indication for use:

Biopsy of bone lesions.



AprioMed

- Gain access into the bone, even through thick cortical bone.
- Facilitate multiple sampling, or treatment of lesions.
- Achieve excellent core samples with few crush artifacts.
- Successfully sample somewhat sclerotic lesions.


ECCENTRIC DRILL TIP

- Facilitates penetration of thick cortical bone
- Drills a hole larger than its' own diameter



3 DIFFERENT LENGTHS

- Allows user to choose the most suitable length for the procedure
- Both Bonopty® sets have 3 different sizes

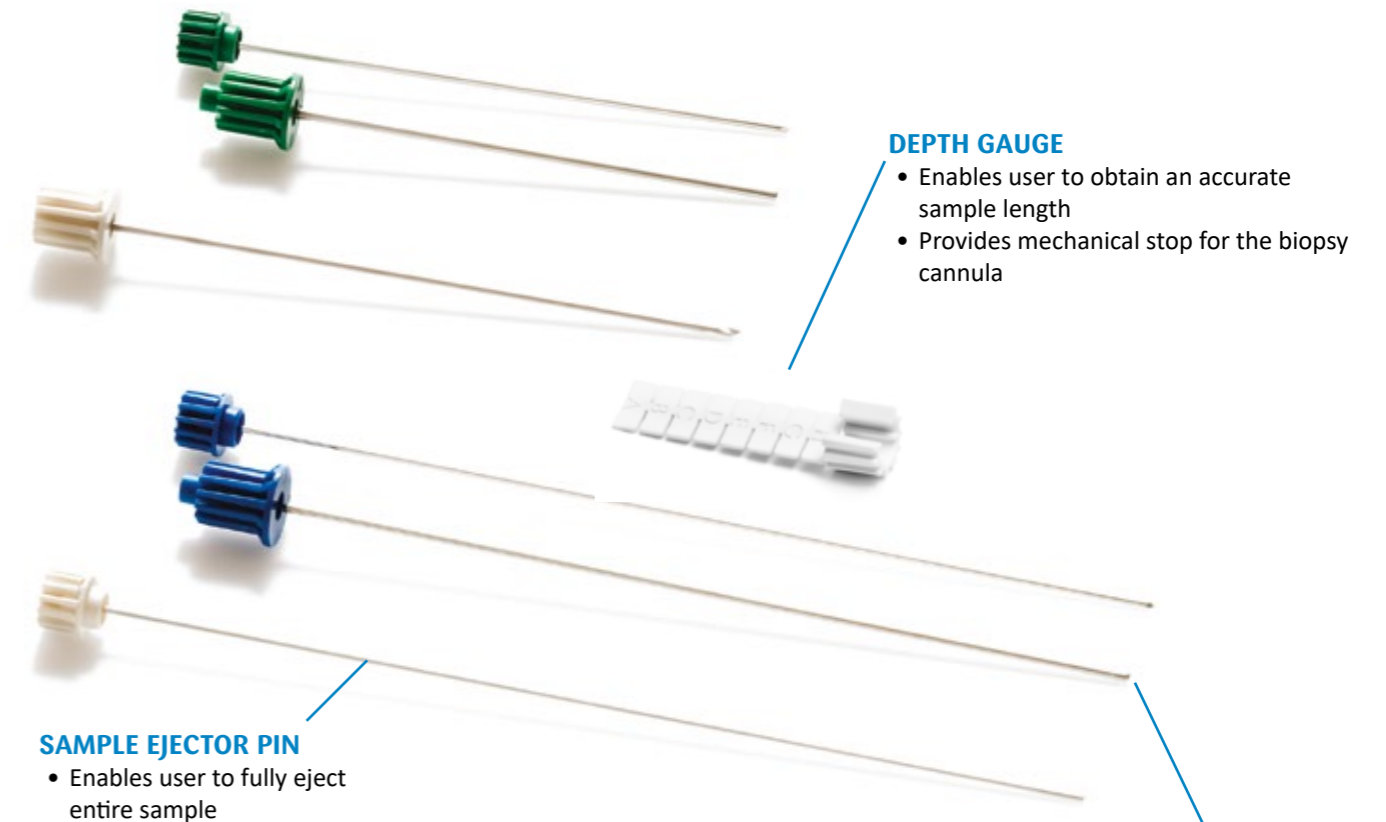

DEPTH GAUGE


- Enables user to obtain an accurate sample length
- Provides mechanical stop for the biopsy cannula

SAMPLE EJECTOR PIN

- Enables user to fully eject entire sample


OUTER REDUCTION OF BIOPSY CANNULA

- Reduces friction during sampling by allowing easy penetration of cortical bone

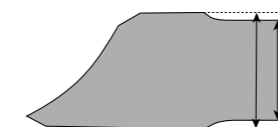


INNER REDUCTION OF BIOPSY CANNULA

- Allows user to obtain larger specimens
- Ensures preservation of specimen structure

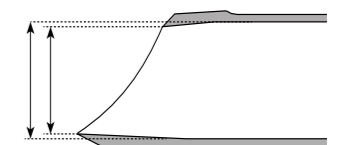

Bonopty® Extended Drill

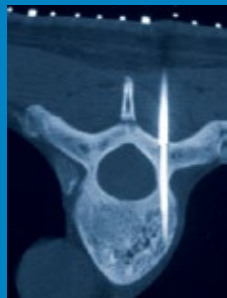

Using Bonopty® Extended Drill you may pass or sample sclerotic lesions deeper in the bone and also do trans-osseous biopsies.

TECHNICAL SPECIFICATIONS

Bonopty® Penetration Set, 14G

Package contents:

- Penetration cannula with stylet
Inner diameter 1.8 mm
Outer diameter 2.1 mm
Length 9.2/9.5/13 cm
- Drill
Diameter 1.7 mm
Length 6.5/12.2/16 cm
- Depth gauge



Bonopty® Biopsy Set, 15G

Package contents:

- Biopsy cannula with stylet
Inner diameter 1.3 mm
Outer diameter 1.7 mm
Length 3/16/19.5 cm
- Ejector pin
- Depth gauge



Bonopty® Extended Drill 15G

Package contents:

- Extended Drill
Diameter 1.7 mm
Length 16 cm



ORDERING INFORMATION

Product number	Product description	Minimum Order Quantity
10-1062	Bonopty® Penetration Set, 14 gauge, 6.5 cm	5 units
10-1072	Bonopty® Penetration Set, 14 gauge, 9.5 cm	5 units
10-1082	Bonopty® Penetration Set, 14 gauge, 13 cm	5 units
10-1063	Bonopty® Biopsy Set, 15 gauge, 13 cm	5 units
10-1073	Bonopty® Biopsy Set, 15 gauge, 16 cm	5 units
10-1083	Bonopty® Biopsy Set, 15 gauge, 19.5 cm	5 units
10-1074	Bonopty® Extended Drill, 15 gauge, 16 cm	5 units



Indication for use: Biopsy of bone lesions.

Before using Bonopty® read the instructions for use which accompany the product for indications, contraindications, warnings and precautions.

Bonopty® is a registered trademark of AprioMed AB. Patents pending.

HEAD OFFICE

AprioMed AB
Virdings Allé 28
SE-754 50 Uppsala, Sweden
Phone: +46 18 430 14 40
Free Fax (Int'l): +46 770 339972

US SALES OFFICE

AprioMed, Inc.
2 Palmer Dr, Suite 1
Londonderry, NH 03053, USA
Phone: +1 603 421 0875
Fax (US toll free): 866 239 1797



info@apriomed.com / www.apriomed.com

APRIOMED develops, manufactures, markets and sells innovative medical devices and related services within the field of interventional radiology. We aim to deliver, in close collaboration with healthcare practitioners, innovative quality tools to achieve the most favourable solutions for radiologists worldwide. AprioMed is headquartered in Uppsala, Sweden, with a sales subsidiary in Londonderry, NH, USA. Our products are sold in more than 40 countries by AprioMed (direct sales in US, UK, Ireland and Sweden) and our local distributors.