

Bonopty[®]

ADAPTER 1:3 REVOLUTIONS

Ergonomic grip for faster drilling

- Transfers and enhances fingertip force to reduce drilling procedure time
- Ergonomic grip ensures full control while preserving tactility and precision
- Universal attachment fits both Bonopty[®] 12g and Bonopty[®] 14g drill



Indication for use

Intended for use in the drilling procedure together with Bonopty[®] Bone Biopsy System




AprioMed

USE OF BONOPTY® ADAPTER 1:3 REVOLUTIONS

1 Start procedure as per normal routine.


a Press penetration cannula with stylet down to bone.

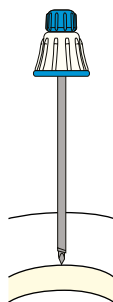


b Replace stylet with eccentric drill.

Rotate a few turns until drill tip cathces the bone.

Verify position and direction using diagnostic imaging.

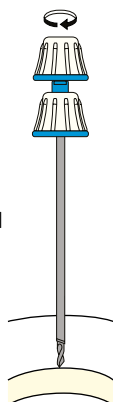

c Retract penetration cannula to expose drill tip beyond penetration cannula to allow the eccentric movement of the drill.

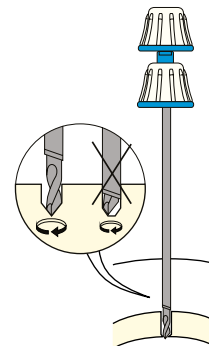


2 Attach Bonopty Adapter 1:3 Revolutions to the drill hub.

Grip the base of the adapter using finger and thumb grip.

Turn the top of the adapter and feel the eccentric drill turning. Use moderate pressure only.

After drill has passed through cortical bone, anchor penetration cannula 2-3 mm into cortex.

Remove drill and adapter.

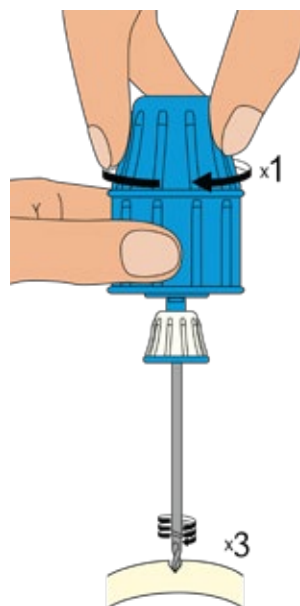

Using Bonopty® Adapter 1:3 Revolutions, one complete turn of the top of the handle...



...corresponds to three turns of the drill tip

3 Insert biopsy needle through the anchored cannula and continue procedure as per normal routine.

ORDERING INFORMATION

Product number	Product description	Minimum Order Quantity
250	Bonopty® Adapter 1:3 Revolutions	5 units

Indication for use: Intended for use in the drilling procedure together with Bonopty® Bone Biopsy System. Before using Bonopty® Adapter 1:3 Revolutions read the instructions for use which accompany the product for indications, contraindications, warnings and precautions. Bonopty® is a registered trademark of AprioMed AB. Patents pending.

HEAD OFFICE

AprioMed AB
Virdings Allé 28
SE-754 50 Uppsala, Sweden
Phone: +46 18 430 14 40
Free Fax (Int'l): +46 770 339972

US SALES OFFICE

AprioMed, Inc.
45 South Main St Unit 2
Derry, NH 03038, USA
Phone: +1 603 421 0875
Fax (US toll free): 866 239 179



info@apriomed.com / www.apriomed.com

APRIOMED develops, manufactures, markets and sells innovative medical devices and related services within the field of interventional radiology. We aim to deliver, in close collaboration with healthcare practitioners, innovative quality tools to achieve the most favourable solutions for radiologists worldwide. AprioMed is headquartered in Uppsala, Sweden, with a sales subsidiary in Londonderry, NH, USA. Our products are sold in more than 40 countries by AprioMed (direct sales in US, UK, Ireland and Sweden) and our local distributors.