


Pelvic lymphocele after kidney transplantation

Procedural Information

Date of procedure: August 2016
Location: Strasbourg, France
Hospital: University Hospital
of Strasbourg
Physician: Julien Garnon, MD


Case Description

in this case, Gangi-SoftGuard[®] was used for drainage of a pelvic lymphocele after kidney transplantation. Use of the spring-loaded blunt tip stylet made it possible to safely pass the needle close to the iliac artery and the femoral nerve, and the drain was positioned without complications.

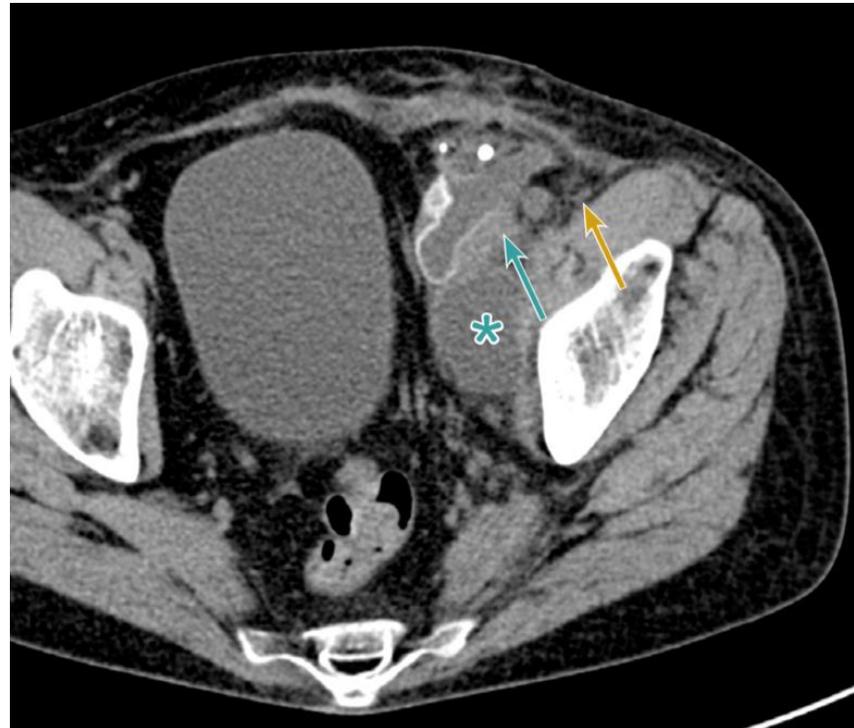


Collection in the pelvic area marked with asterisk. The needle pathway to target is close to the iliac artery (turquoise) and the femoral nerve (yellow).

Pelvic lymphocele after kidney transplantation

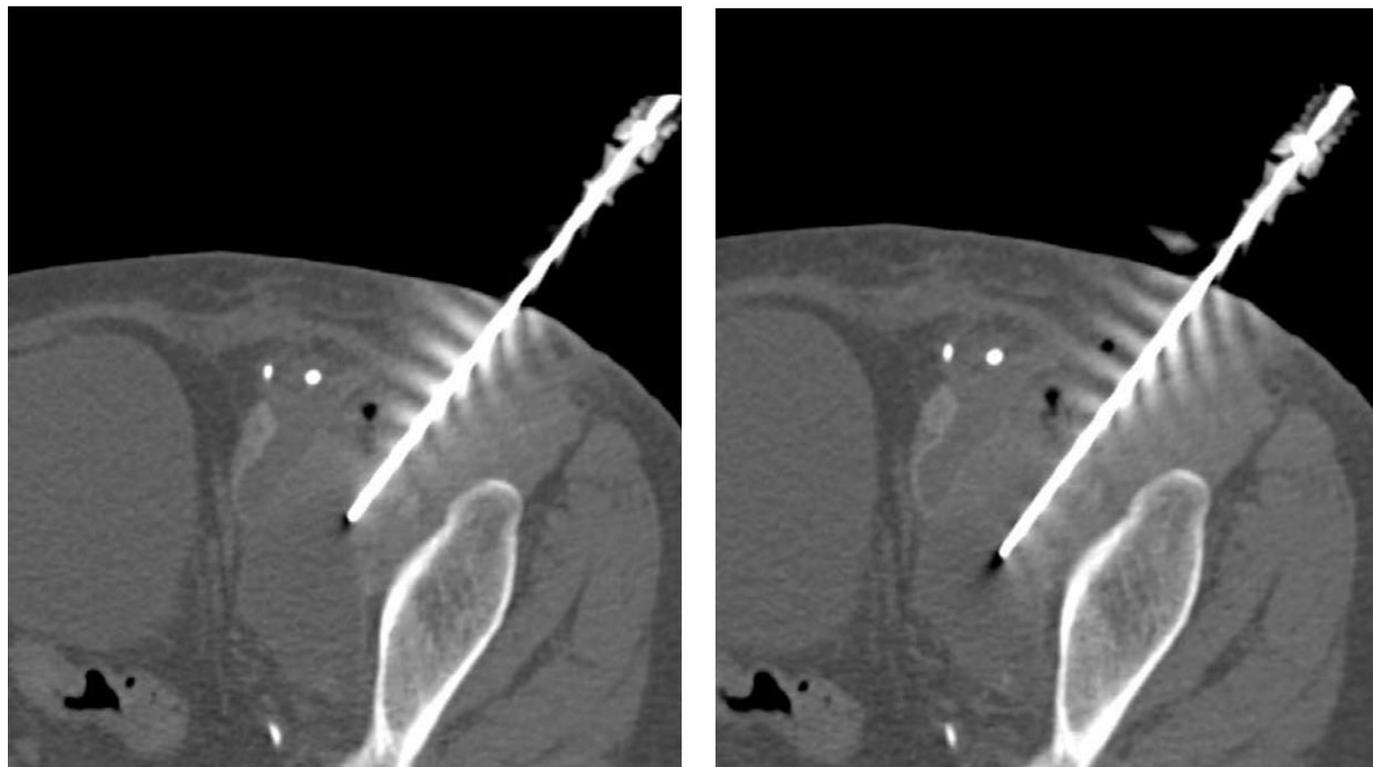

Gangi-SoftGuard[®] is inserted and advanced to the collection.

Pelvic lymphocele after kidney transplantation

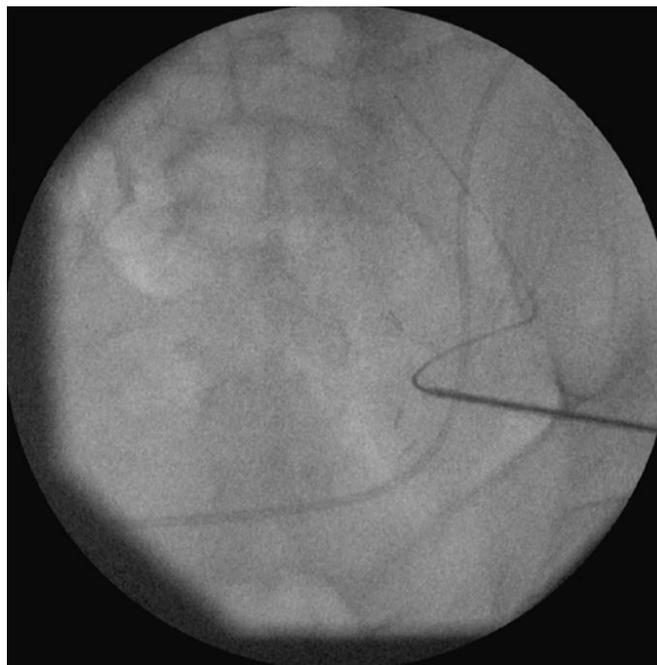

Guidewire is inserted through the Gangi-SoftGuard® needle.



Drain is in position.

Case and image courtesy of Julien Garnon, MD, University Hospital of Strasbourg, Strasbourg, France