


AprioCore[®] *plus*

SEMI-AUTOMATIC BIOPSY SYSTEM

Step by step description




Step 1 - Load

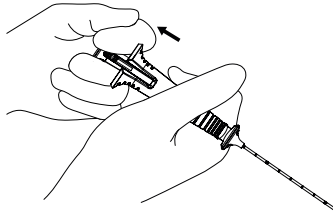

Prior to insertion of AprioCore[®] plus, load the device by pulling the trigger button until a firm click is felt, indicating that the device is ready for use.


Confirm that the stylet is fully retracted and that the specimen release button is in the rearmost position.

Step 2 - Advance

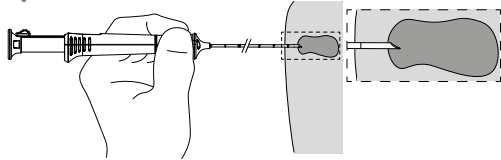


Advance the needle to the area to be biopsied. Do not advance the slotted stylet until AprioCore[®] plus is in position.

Step 3 - Expose notch

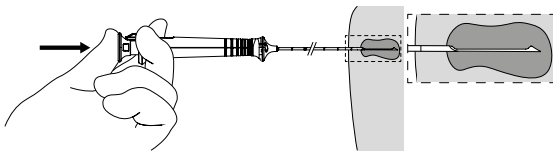


While maintaining needle position, advance the slotted stylet with thumb to expose the specimen notch within the area to be biopsied.

Step 4 - Capture

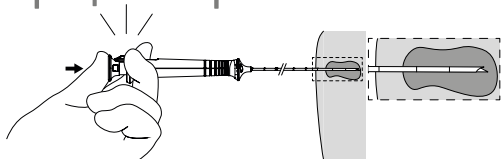


Confirm that the specimen release button is in its retracted position. Press the trigger button past the first stop to capture the tissue specimen.

Step 5 - Release

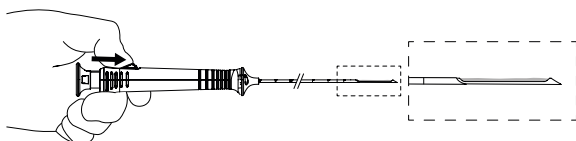

To remove the tissue specimen, push the specimen release button forward to expose the specimen notch.

 Indication for use: AprioCore[®] plus is intended for use in obtaining core biopsy specimens from soft tissue such as breast, kidney, liver, lung, thyroid, lymph nodes and various soft tissue masses performed under image guidance techniques for soft-tissue biopsy. Before using AprioCore[®] plus read the instructions for use which accompany the product for indications, contraindications, warnings and precautions. AprioCore[®] plus is a registered trademark of AprioMed AB. Patents pending.

HEAD OFFICE

AprioMed AB
Virdings Allé 28
SE-754 50 Uppsala, Sweden
Phone: +46 18 430 14 40
Free Fax (Int'l): +46 770 339972

US SALES OFFICE

AprioMed, Inc.
45 South Main St Unit 2
Derry, NH 03038, USA
Phone: +1 603 421 0875
Phone (US toll free): +1 866-230-1323
Fax (US toll free): +1 866-239-1797



info@apriomed.com / www.apriomed.com